




www.neatstitch.com
info@neatstitch.com

2020006 V1



NeatClose Parts

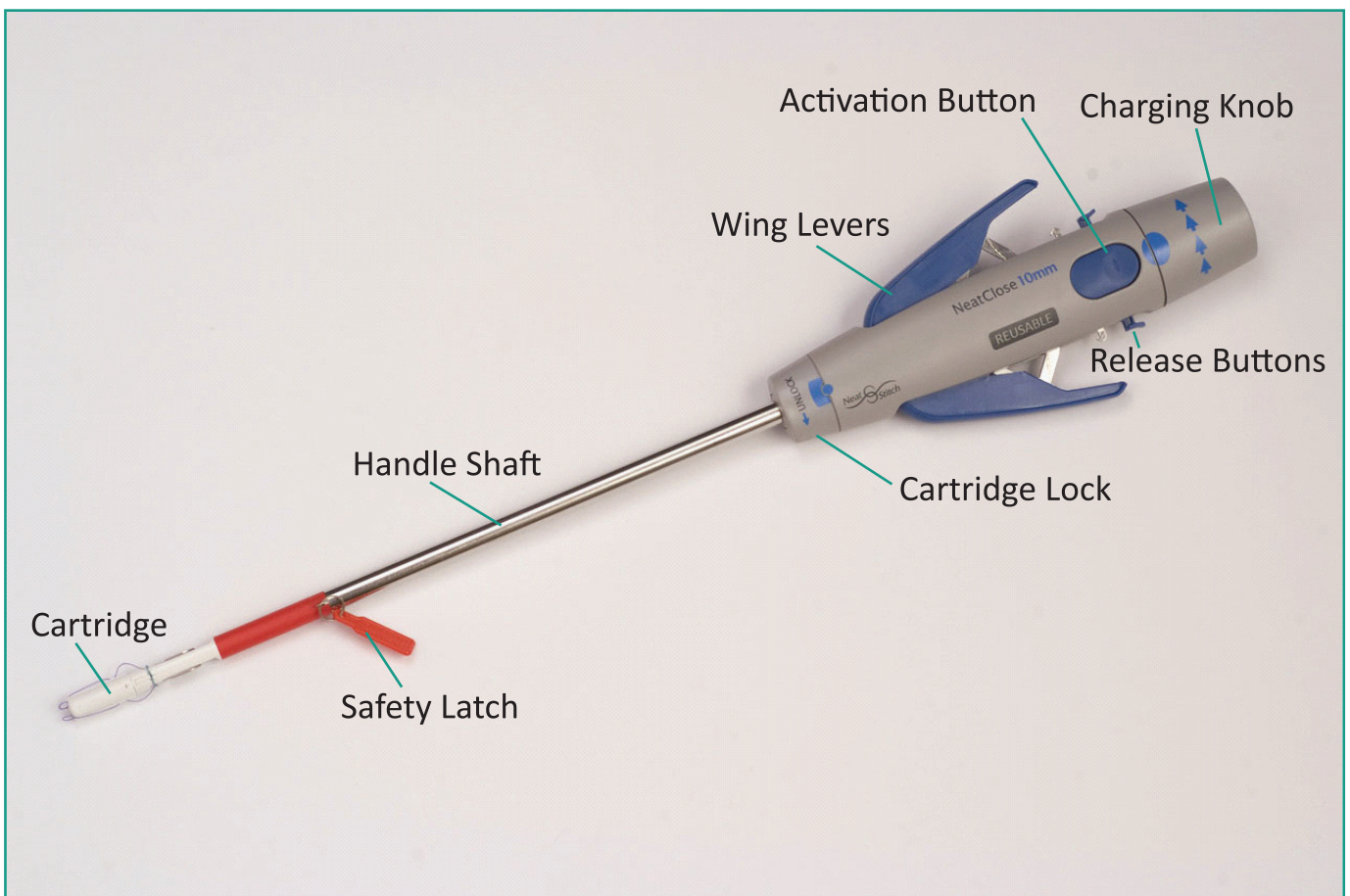

NeatClose Preparation

Nurse

1 Open Wing Levers

- Squeeze** Levers with your left hand.
- Press** the two blue Release Buttons with your right hand
- Release** the Levers by slowly opening your left hand.



2 Wind the Charging Knob

- Press** Release Buttons with your left hand.
- While pressing both buttons, begin **rotating** Charging Knob with your right hand.
- After beginning rotation, release buttons and move your left hand to the base of the handle body and complete a full rotation (360°) of the Charging Knob until you hear a **click**.



3 Insert Cartridge

- Unlock** the Cartridge Lock by turning it in the direction of the arrow.
- Insert** the Cartridge into the Handle Shaft until it is fully situated in the metal tube. Rotate the cartridge gently if necessary.



4 Lock Cartridge

- Turn** the Cartridge Lock back to its original position.



5 Remove Safety Latch

- Pull** and remove the Safety Latch from the Cartridge



The device is now ready for the surgeon's use

NeatClose Operation

Surgeon

1 Insert Device

Insert the NeatClose™ device into the port until the red part of the Cartridge is visible via the endoscope.



2 Squeeze Wings Levers

Squeeze both Handle Wing Levers until both Levers are closed and locked. The Needle Guides are now deployed.




3 Slide the Trocar up

Slide the trocar cannula all the way up the handle shaft



4 Position the Needle Guides

Withdraw the NeatClose™ device to position both Needle Guides in firm but gentle contact with the peritoneum and perpendicular to the incision line. The Handle should also be perpendicular to the body surface. **Do not** rotate the handle after positioning the Needle Guides.

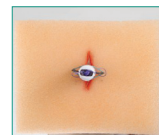



5 Activate NeatClose™

Keep the Handle stationary and **Push** the blue activation button to fire the needles.



6 Withdraw Handle

Slowly **withdraw** the Handle from the port, fully releasing the threads from the cartridge. **Cut** the threads and **tie** routinely.

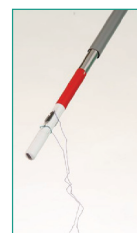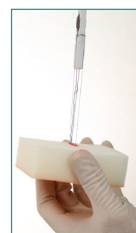

Unlock Cartridge Lock to remove the used cartridge